


ACTIV CARE ADJUSTABLE BED

FIXED HEADBOARD INSTALLATION GUIDE

FIXED HEADBOARDS


Allure Fixed Headboards

Allure Long Single	BEA830150
Allure King Single	BEA830152
Allure Long Double	BEA830154
Allure Queen	BEA830156

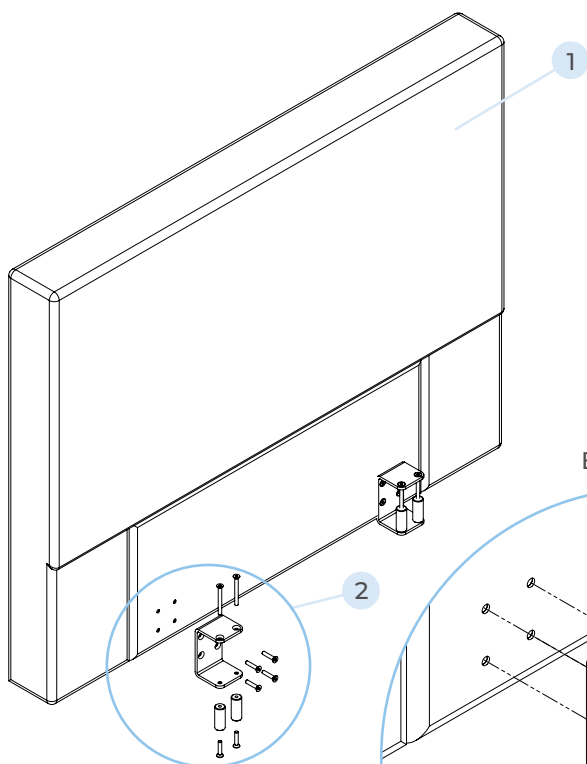

ITEM	DESCRIPTION	QTY
1	Allure Fixed Headboard	1
2	Bracket Kit*	2 - 4
3	M6 x 60 Bolt	2
4	Bracket	1
5	M6 x 20 Bolt	6
6	Spacer	2

Bracket Kit:

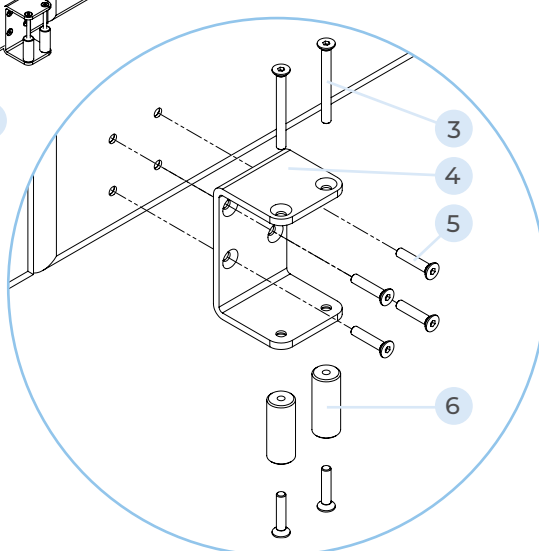

*Number of Bracket Kits included:


x2: Long Single & King Single

x4: Long Double & Queen

HOW TO ASSEMBLE FIXED HEADBOARDS

The Comfimation fixed headboard is designed to be used with all functions of the Comfimation bed. A safety function has been built into the headboard, when the Trendelenburg function is used the headboard will pivot forward with the wall.

TOOLS REQUIRED

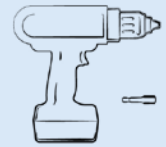


Impact driver
or drill (4mm hex bit)



4mm Allen Key (x2)


For use on the:
✓ ACTIV CARE BED

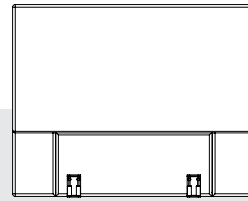

It is recommended that assembly is carried out by TWO persons.

Care must be taken when using trendelenburg function. The headboard will pivot forwards when trendelenburg function is used and the headboard is obstructed by a wall.

- 1 Fix brackets to the headboard.
There are 4x M6 screws per bracket.

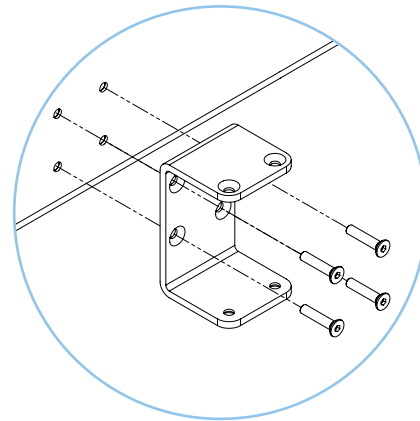


- 2 Raise the backrest on the Comfimation bed to access the bed frame.

NOTE:
Beds with PO01145529 and later in their serial number will have pre drilled holes for mounting.

Beds without mounting holes pre-drilled can have holes drilled by an experienced and competent person. Mark hole positions after step 3, remove the headboard and drill 6.5mm holes through the frame.




BACKREST
Raise to Max

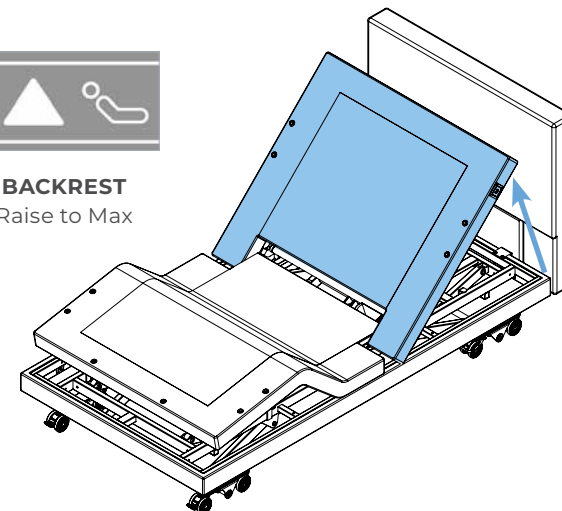


- 3 Position the fixed headboard at the head end of your Comfimation bed.


The C-Shaped bracket is to be positioned around the upholstery and bed frame.

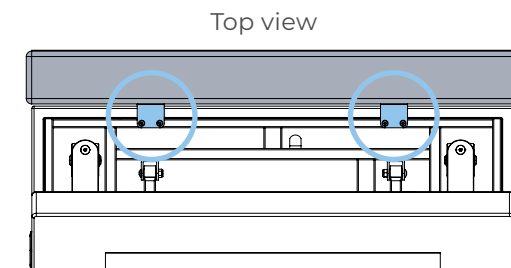


← Headboard
← Brackets
← Bed frame
← Raised backrest

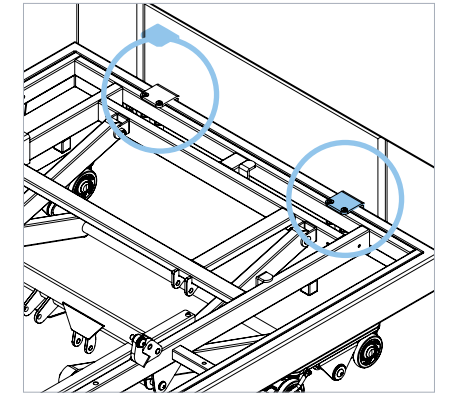


- 4 Fix the spacers to the bottom of the bracket once in position with 2x M6 x 20 screws.

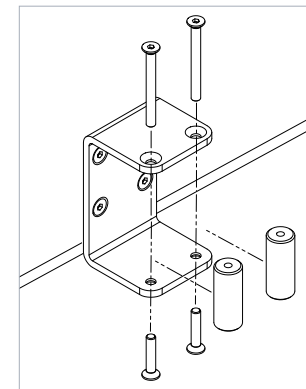

- 5 Align to mounting holes in frame and tighten both top and bottom bolts to secure brackets in place. Make sure the headboard is secure with minimal movement between bracket and frame.

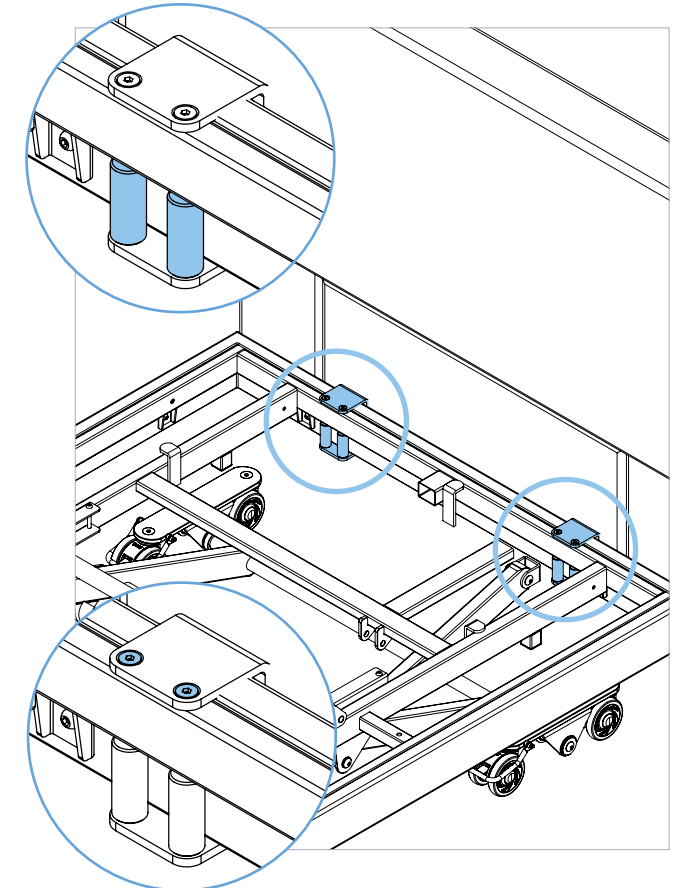

- 6 Position bed with 80mm spacing from the wall to give enough space for the Trendelenburg movement of the bed.

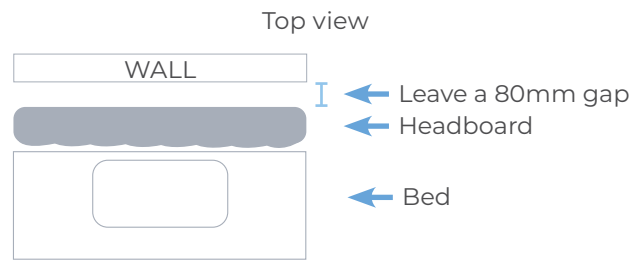

- 7 Test bed with all functions to ensure the headboard is not loose.

